RUS

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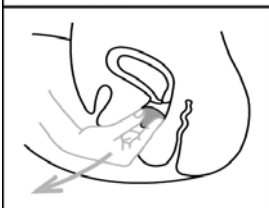
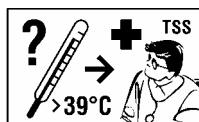
MADE IN GERMANY
Original SexMAX-Produkt

Vertrieb / distribution:
JOYDIVISION international AG
Hannover (Germany)

©

www.SexMAX.eu

GB

Instructions of Use for Hot Lady Sex Tampons

More and more women swear by the **Hot Lady Sex Tampons**: They make sex during a period a relaxing and clean pleasure – no more worries about bedclothes and mattress or about the partner's unease. Very often he doesn't even notice that SHE has her period.

The thread-free **Hot Lady Sex Tampons** are also ideal for the sauna, sport or for swimming. In earlier times women, in part, used natural sponges, which naturally in no way met the hygienic requirements and could cause serious illness and infections. The **Hot Lady Sex Tampons** were developed according to the most up to date gynaecological science. Interior irritations are excluded and the dermatological tolerance is assured! **Hot Lady Sex Tampons** are packed individually and hygienically and cannot be felt by either partner when placed correctly at the neck of the womb.

Absorbability is above average! No contraceptive.

This is how to use the tampon: • Wash your hands and remove the hygienically packed **Hot Lady Sex Tampon**. • For easier insertion the tampon can be moistened with a little **AQUAGlide, Super Rutscher** or with clean water. • After moistening the tampon, squeeze it out a little. This is how it will soak up the most liquid later. • Raise your leg (e.g. on the edge of the bath tub or on a stool) or go in a crouching position (it shortens the way to the neck of the womb from 7-8 cm to about 4.5 cm). • Place the index finger on a flat side of the tampon, grip the tampon with the thumb and middle finger and push it deep enough into the vagina, through the labia that you are holding open with your other hand, that the other flat side of the tampon is placed right at the top at the end of the vagina (in front of the neck of the womb).

Wearing time: To adapt to the heaviness of the period, on heavy days it is recommended that it be changed after 3-5 hours, on less heavy days you can wear it for about 5 to 8 hours. It is recommended that you change the tampon immediately after sex without a condom.

Removing the tampon: Firstly get into a crouching position or sit on the toilet the other way round (e.g. with bent legs). Remove the tampon with a pincer grip (with the thumb and index finger or with the index and middle fingers). (see illustration below).

Should you be unable to remove a tampon, please go to your gynaecologist. Also you should consult your gynaecologist prior to use, if you use a coil with slightly extended retrieval strings as a contraceptive.

Please use each tampon only once, because otherwise the hygienic safety and the high level of absorbability will no longer be assured and it could lead to infection.

As with other hygiene articles for periods, the **Hot Lady Sex Tampons** should be disposed of via the household rubbish. Please recycle the packaging.

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